firesafe™ nozzle

instructions for use

5	EN	Firesafe™ Nozzle - Instructions for Use
6	DE	Firesafe™ Nozzle - Bedienungsanleitung
7	FR	Firesafe™ Nozzle - Mode d'emploi
8	ES	Firesafe™ Nozzle - Instrucciones de uso
9	PT	Firesafe™ Nozzle - Instruções de utilização
10	IT	Firesafe™ Nozzle - Istruzioni per l'uso
11	NL	Firesafe™ Nozzle - Gebruiksaanwijzing
12	EL	Firesafe™ Nozzle - Οδηγίες Χρήσης

REF 827-0030

DISS (CGA V-5 1240, 9/16" UNF)

REF 827-0020

Universal (1/2" x 16 TPI BSF)

EC REP

Qarad EC-REP BV
BE-AR-000000040
 Pas 257, 2440 Geel
 Belgium

CH REP

Qarad Suisse S.A.
CHRN-20002058
 World Trade Center
 Avenue Gratta-Paille 2
 1018 Lausanne
 Switzerland

MedEnvoy
NL-IM-000000248
 Prinses Margrietplantsoen 33
 Suite 123, 2595 AM, The Hague
 The Netherlands

EN Figure 1

DE Abbildung 1

FR Figure 1

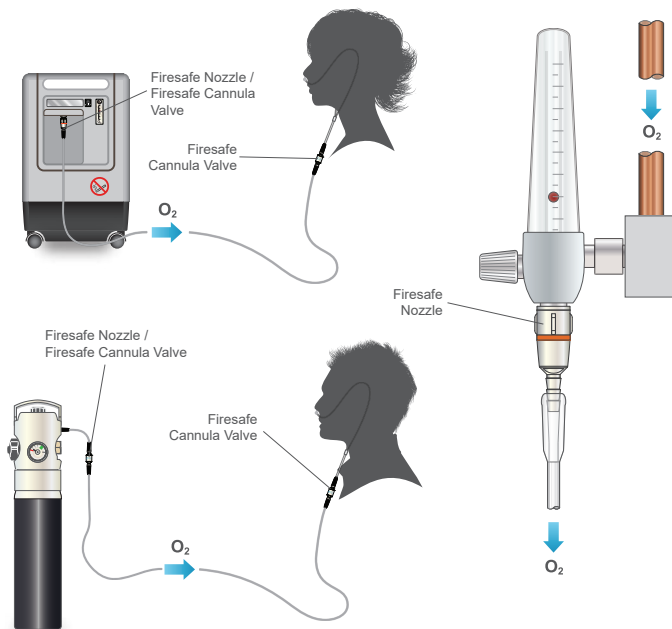
ES Figura 1

PT Figura 1

IT Figura 1

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CE
1639

UK
CA
0120

Intended Use

The Firesafe Nozzle is intended to connect oxygen tubing to a source of supply for use during normobaric oxygen therapy. A further non-medical purpose is to extinguish a fire in oxygen tubing and stop the flow of oxygen.

Precautions

1. This device offers a resistance to flow; consider the impact of this on the system when installing the device.
2. If the flow is blocked downstream, the pressure will rise to the supply system maximum pressure. It is recommended that a type test is carried out to ensure connections can withstand this pressure.
3. For oxygen concentrator applications, to meet the requirements of ISO 80601-2-69, the DISS version of the Firesafe Nozzle (REF 827-0030) should be tightened to a torque of 3 Nm (2.2 lbf.ft). This will ensure the Firesafe Nozzle can only be removed by the use of a tool.
4. Do not immerse the device in any fluid or allow fluid to enter the device.
5. If the packaging is damaged and unsealed or it has been exposed to temperatures outside the transport and storage temperature range, the devices should not be used.

Warnings

1. Do not administer oxygen therapy when smoking or when near sources of heat, flame or sparks.
2. Keep this device free from oil and/or grease. Do not lubricate the hose barb.
3. Additional risk control measures (e.g. SpO2 monitoring with an alarm) are essential for patients critically dependent on oxygen therapy.

Installation Instructions

1. Remove the existing connector (as applicable) from the oxygen supply source and fit the appropriate Firesafe Nozzle.
2. Check that gas flows freely through the Firesafe Nozzle and that there are no leaks.

If any serious incident that occurs relating to the device, it should be reported to BPR Medical and the relevant regulatory authority where the incident occurred.

Cleaning, Maintenance & Disposal

The Firesafe Nozzle is a non-sterile, reusable medical device that does not require reprocessing between uses. Clean the exterior surfaces of the device with mild detergent solution or disinfectant wipe. The Firesafe Nozzle is maintenance free. If actuated by a fire, it cannot be reused and should be disposed of. There are no special considerations for disposal.

Device Specification

Resistance to Flow	0.2 kPa at 2 l/min
	0.9 kPa at 5 l/min
	6.8 kPa at 15 l/min
Flow rate ⁽¹⁾	1 - 25 l/min
Maximum operating pressure ⁽²⁾	1000 kPa
Operating temperature	0 to 50 °C (32 to 122 °F)
Transit & storage temperature range	-25 to 70 °C (-13 to 158°F)
Intended life ⁽³⁾	8 years

⁽¹⁾ At flows less than 1 l/min, a fire will be extinguished but the device may not shut off the oxygen flow.

⁽²⁾ Maximum static pressure delivered by the gas supply source.

⁽³⁾ An expiry date  is engraved on the device.

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