firesafe™ cannula valve  
instructions for use

6	EN	Firesafe™ Cannula Valve - Instructions for Use
7	DE	Firesafe™ Rückschlagventil – Bedienungsanleitung
8	FR	Vanne de canule Firesafe™ - Mode d'emploi
9	ES	Válvula para cánulas Firesafe™ - Instrucciones de uso
10	IT	Valvola per cannula Firesafe™ - Istruzioni per l'uso
11	NL	Firesafe™ Cannula Valve - Gebruiksaanwijzing
12	PT	Válvula de cânula Firesafe™ - Instruções de utilização
13	SV	Firesafe™ Slangventil - Användarinstruktioner
14	CZ	Kanylový ventil Firesafe™ - Pokyny k použití
15	PL	Zawór kaniuli Firesafe™ - Instrukcja obsługi
16	RU	Клапан для каниули Firesafe™ - Инструкции по эксплуатации
17	RO	Valva pentru canulă Firesafe™ - Instrucțiuni de utilizare
18	FI	Firesafe™ kanyyliventtiili - Käyttöohje
19	DA	Firesafe™ Cylinderventil - Brugsvejledning
20	SK	Kanylový ventil Firesafe™ - Návod na použitie
21	NO	Firesafe™ kanyleventil - Bruksanvisning
22	SL	Ventil Firesafe™ za kanilo - Navodila za uporabo
23	LT	Firesafe™ kaniulės vožtuvas - Naudojimo instrukcija
22	LV	Firesafe™ kanulas vārsts - Lietošanas instrukcija
23	HU	Használati útmutató Firesafe™ - kanülszelep
24	EL	Βαλβίδα Κάνουλας Firesafe™ - Οδηγίες Χρήσης
25	ET	Firesafe™ kanüüli klapp - Kasutusjuhend
26	BG	Каниола с клапан Firesafe™ - Инструкции за употреба
27	HR	Ventil kanile Firesafe™ - Upute za uporabu

**QbD RepS BV**  
**BE-AR-000000040**  
 Groenenborgerlaan 16  
 2610 Wilrijk, Belgium


**MedEnvooy**  
**NL-IM-000000248**  
 Prinses Margrietplantsoen 33  
 Suite 123, 2595 AM, The Hague  
 The Netherlands


**Qarad EC-REP BV**  
**CHRN-20002058**  
 World Trade Center  
 Avenue Gratta-Paille 2  
 1018 Lausanne, Switzerland

EN	Figure 1	PT	Figura 1	FI	Kuvan 1	LV	Attēla 1
DE	Abbildung 1	SV	Figur 1	DA	Figur 1	HU	Ábra 1
FR	Figure 1	CZ	Obrázku 1	SK	Obrázku č. 1	EL	Σχήμα 1
ES	Figura 1	PL	Rysunku 1	NO	Figur 1	ET	Joonisel 1
IT	Figura 1	RU	Рисунке 1	SL	Sliki 1	BG	Фигура 1
NL	Afbeelding 1	RO	Figura 1	LT	Pav 1	HR	Slici 1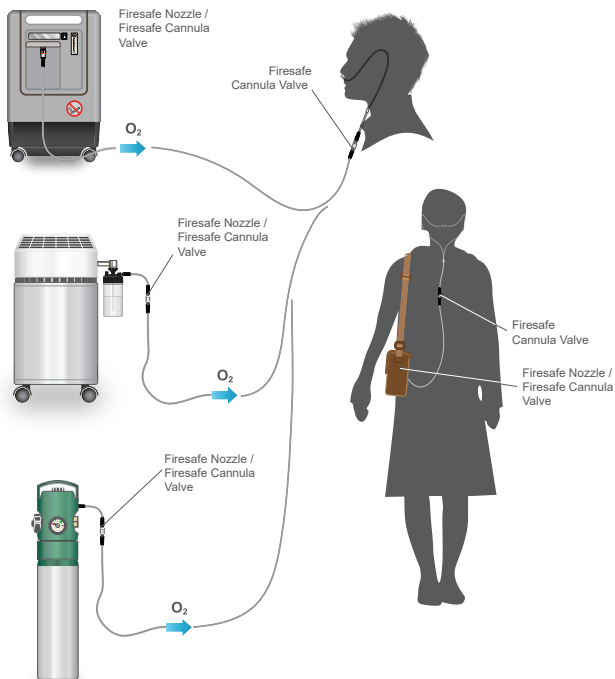

EN	Figure 2	PT	Figura 2	FI	Kuvan 2	LV	Attēla 2
DE	Abbildung 2	SV	Figur 2	DA	Figur 2	HU	Ábra 2
FR	Figure 2	CZ	Obrázku 2	SK	Obrázku č. 2	EL	Σχήμα 2
ES	Figura 2	PL	Rysunku 2	NO	Figur 2	ET	Joonisel 2
IT	Figura 2	RU	Рисунке 2	SL	Sliki 2	BG	Фигура 2
NL	Afbeelding 2	RO	Figura 2	LT	Pav 2	HR	Slici 2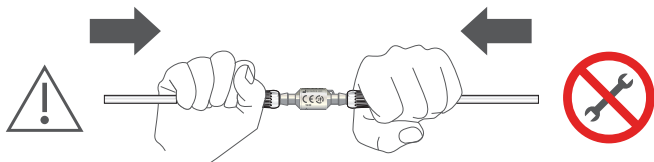

## Intended Use

The Firesafe Cannula Valve is intended to connect to oxygen tubing and oxygen delivery accessories for use during normobaric oxygen therapy in the home healthcare environment and for ambulatory use. A further non-medical purpose is to extinguish a fire in oxygen tubing and stop the flow of oxygen.

## Precautions

1. This device offers a resistance to flow; consider the impact of this on the system when installing the device.
2. If the flow is blocked downstream, the pressure will rise to the supply system maximum pressure. It is recommended that a type test is carried out to ensure connections can withstand this pressure.
3. Do not immerse the device in any fluid or allow fluid to enter the device.
4. If the packaging is damaged and unsealed or it has been exposed to temperatures outside the transport and storage temperature range, the devices should not be used.

## Warnings

1. Do not administer oxygen therapy when smoking or when near sources of heat, flame or sparks.
2. Keep this device free from oil and/or grease. Do not lubricate the hose barb.
3. Additional risk control measures (e.g. SpO2 monitoring with an alarm) are essential for patients critically dependent on oxygen therapy.
4. Only respiratory therapy tubing and connectors complying with ISO 17256 should be used with this device.
5. Fire can arise from the use of lubricants not recommended for use during oxygen therapy.
6. Use of this device does not prevent fires; the risk of burn injury is still present as fabrics saturated with oxygen can still ignite and burn vigorously.
7. This device does not stop the flow of oxygen if the oxygen tubing is ignited between it and the gas supply.

## Installation Instructions

1. Firmly connect oxygen tubing connectors to both ends of the device.
2. Check that gas flows freely through the Firesafe Cannula Valve and that there are no leaks.

Note: To ease replacement of a nasal cannula or mask, the connection closest to the patient should not be connected too tightly.

If any serious incident that occurs relating to the device, it should be reported to BPR Medical and the relevant regulatory authority where the incident occurred.

## Cleaning, Maintenance & Disposal

The Firesafe Cannula Valve is a non-sterile, single patient multiple-use medical device. Clean the exterior surfaces of the device with a mild detergent or disinfectant wipe. The Firesafe Cannula Valve is maintenance free. If actuated by a fire, it cannot be reused and should be disposed of. There are no special considerations for disposal.

It is not possible to inspect, clean or disinfect the device internally. Reprocessing the device for use with the new patient could result in damage to the device or failure to remove contaminants.

## Device Specification

Flow rate <sup>(1)</sup>	0.25 - 25 l/min
Maximum operating pressure <sup>(2)</sup>	600 kPa
Operating temperature	0 to 50 °C (32 to 122 °F)
Transit & storage temperature range	-25 to 70 °C (-13 to 158 °F)
Intended life <sup>(3)</sup>	5 years

<sup>(1)</sup>At flows less than 0.25 l/min, a fire will be extinguished but the device may not shut off the oxygen flow.

<sup>(2)</sup>Maximum static pressure delivered by the gas supply source.

<sup>(3)</sup>An expiry date  is engraved on the device.

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